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# Settings

←

# Q Search

Advanced Settings	
Арр	
Find My Scanner	
To keep track of Scanner in case of loss, record the location of each exam. Review Clarius Privacy Policy.	
Auto Connect	
Automatically connect to the last used Scanner upon launching App (battery must be plugged into Scanner).	
Auto Scan on Start	
Automatically start imaging instead of launching exam worksheet.	
Auto Scan Post Cine	
Automatically continue scanning after capturing a prospective cine.	
Auto Capture Measurements	
When unfreezing, automatically store an image if measurements are on the screen.	
Auto Exam Upload	
After ending an exam, disconnect from the Scanner and immediately upload to Cloud or DICOM.	
Offset Calipers	
Offset measurement calipers between touch locations.	
Control Orientation	
Align controls to the right side of the screen.	
Prioritize Spectrums	
Prioritize spectral displays for maximium visibility	

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# Settings

Q Search	
Advanced Settings	
Spectrum Tap	•
Forces a tap on the 2D image or PW spectrum to change acquisition mode when duplex imaging cannot be engaged (mono PW).	
Advanced Imaging Controls	
Display advanced imaging controls in the menus.	-
Announcements	
Display new announcements when App starts.	-
ProTips	
Display ProTips while connecting.	_
Virtual Scanners	
Show virtual Scanners for demonstration purposes.	
Cine Capture Length	3 🔻
The number of seconds for a prospective cine capture and the default length of a retrospective cine capture.	
Exam Auto Purge	10 🔻
The number of days before an exam is automatically deleted after successful upload to the Clarius Cloud or to a DICOM Storage served	/er.
Classroom Auto Purge	10 🔻
The number of days before a classroom is automatically deleted after it has been consulted at least once.	
Long Press	Auto Caliper 🔻
Action to perform when long pressing on a frozen image.	
Language Settings	American English (United States) 🔻

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Display vertical scale alongside image.

#### Measurements

Overlay measurement calipers and measured values.

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## Header Fields

←

Select which fields will appear in a header atop exported images.

Patient Name	
Patient ID	
Date of Birth	
Gender	
Institution Name	
Institution Address	
Exam Date	
Operator ID	•
Scanner Model	
Application	

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Select which information should appear in a footer under exported images.

### **Doppler Parameters**

## **DICOM Export**

### Transfer Syntax

Transfer Syntax for DICOM files stored locally. Note: this option will alter the size of the files of stored.

### Structured Report

Export the DICOM Structured Report (DSR) with local DICOM exports.

## Local Export

#### Export Location

Location where images will be exported. Note: This does not affect the export location of PDF, DICOM, or Raw Data which are always exported to Files.

## **PDF Reports**

### Image Grid

Grid layout for images contained within a report.

### Export/Share PDF with Exams

Automatically create and export PDFs when exporting locally once the exam is complete, or when sharing through third-party Apps.

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# Settings

Q Search	
dvanced Settings	
Mobile Device	
Auto Brightness Automatically set mobile device screen to maximum brightness while App is running.	-
Blue Tint Generate images with a blue tint.	
Scanner	
Auto Boot	•
Disable Sound Prevent the Scanner from making sounds.	•
Contact Detection	3 •
The number of seconds to engage a lower frame rate mode when no contact has been detected.	
Auto Freeze	30 🔻
The number of seconds to automatically freeze once the Scanner has been imaging without patient contact.	
Keep Awake	15 💌
The number of minutes after the Scanner has been frozen before shutting down.	
Button Settings	

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# Button Options (HD Scanners Only)

 $\leftarrow$ 

Disable Power Button	•
Prevent the buttons on the scanner from turning on the scanner.	
Button Up	User Defined 🔻
Select the function of the scanner's up button.	
Single Click	Capture Image 🔻
Select the function performed when the scanner's up button is clicked.	
Double Click	Disabled 🔻
Select the function performed when the scanner's up button is clicked twice.	
Triple Click	Disabled 🔻
Select the function performed when the scanner's up button is clicked three times.	
Quad Click	Disabled 🔻
Select the function performed when the scanner's up button is clicked four times.	
Button Down	User Defined 🔻
Select the function of the scanner's down button.	
Single Click	Freeze 🔻
Select the function performed when the scanner's down button is clicked.	
Double Click	Disabled 🔻
Select the function performed when the scanner's down button is clicked twice.	
Triple Click	Disabled 🔻
Select the function performed when the scanner's down button is clicked three times.	
Quad Click	Disabled 🔻
Select the function performed when the scanner's down button is clicked four times.	
Button Up (Hold)	Disabled •

Select the function when the scanner's up button is held for 3 seconds.

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Keep Scanner awake while charging.

#### Image On Undock

Automatically start imaging when the scanner is removed from the charger.

#### Alarm On Undock

Engage a visual and audible alarm when scanner is removed from the charger.

## Connectivity

IMPORTANT: Clarius Scanners rely on the mobile device's region configuration to convey proper regulatory domain information. The Scanners will automatically limit use to the regulated wireless bands based on the mobile device's configured region.

#### Bandwidth Optimization

Allow bandwidth optimization to adjust parameters such as frame rate and compression based on real-time connectivity parameters.

#### Wi-Fi Optimization

Allow the scanner to detect poor Wi-Fi scenarios, which stops scanning to present a connectivity resolution option.

### Prefer Wireless LAN

Connect via wireless LAN (via mobile device's current network), if available. WARNING: Enabling this option may impact performance as wireless LANs may have other network traffic.

### Wi-Fi Search

Search for the best Wi-Fi channel on Scanner boot. WARNING: Enabling this option may slow down Scanner bootup by up to 5 seconds, but may yield better connectivity results.

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4

## Settings

Q Search	
Advanced	ettings
Connec	ivity Resolution Quick Resolve 🔻
Action to up to 15	ake when connectivity performance is sub-optimal. IMPORTANT: The Quick Resolve setting is the fastest and simplest resolution method, whereas Simple Search may take econds, but may yield better results in an area with many Wi-Fi networks.
	Reset Wi-Fi
SSID	Customization
0	SSID Name
ô	Password
	Configure

## **Motion Sensor**

### Auto Wake

Automatically start imaging when Scanner is picked up (HD Scanners only).

### Auto Freeze

The number of seconds to automatically freeze once the Scanner has become stationary.

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 $\leftarrow$ 

 Advanced Settings

 Foot Pedals
 Left Pedal Function
 The function to perform in response to the left foot pedal.
 Right Pedal Function
 Capture Image •

 The function to perform in response to the right foot pedal.

 Foot Pedal Selection

## **Screen Calibration**

Adjust the Calibration Bar to ensure all grayscale level numbers can be evenly distinguished. The recommended setting is 12.

## **Reset Options**

## **Applications Reset**

WARNING: This will reset all saved customizations (such as depth, gain, etc.) applied to applications.

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